

**DANVILLE****ENAMEL BOND****E-BOND™****INSTRUCTIONS**

E-Bond™ is a light cure, bis-GMA based bonding agent, a typical “enamel bond”. It is used for bonding to etched enamel and for bonding to silane treated porcelain. It is also an excellent lubricant to prevent sticking of composites to instruments.

**DIRECTIONS FOR USE**

After etching of enamel with a phosphoric acid such as Sure Etch™, apply the E-Bond with a brush or other applicator. It does not require thinning since it does not contain any solvent. Light cure and proceed with composite application.

**STORAGE**

The shelf life of E-Bond™ is 2 years when stored at 77° F (25° C) or below.

 Danville Materials  
a Zest Anchors, LLC company  
2875 Loker Avenue East  
Carlsbad, CA 92010 USA  
(1)760-743-7744

[www.zestdent.com](http://www.zestdent.com)

**ZD ZEST DENTAL**  
SOLUTIONS

 MDSS GmbH  
Schiffgraben 41  
30175 Hannover  
Germany



89109-00 - REV F

**DANVILLE****ENAMEL BOND****E-BOND™****INSTRUCTIONS**

E-Bond™ is a light cure, bis-GMA based bonding agent, a typical “enamel bond”. It is used for bonding to etched enamel and for bonding to silane treated porcelain. It is also an excellent lubricant to prevent sticking of composites to instruments.

**DIRECTIONS FOR USE**

After etching of enamel with a phosphoric acid such as Sure Etch™, apply the E-Bond with a brush or other applicator. It does not require thinning since it does not contain any solvent. Light cure and proceed with composite application.

**STORAGE**

The shelf life of E-Bond™ is 2 years when stored at 77° F (25° C) or below.

 Danville Materials  
a Zest Anchors, LLC company  
2875 Loker Avenue East  
Carlsbad, CA 92010 USA  
(1)760-743-7744

[www.zestdent.com](http://www.zestdent.com)

**ZD ZEST DENTAL**  
SOLUTIONS

 MDSS GmbH  
Schiffgraben 41  
30175 Hannover  
Germany



89109-00 - REV F