

DANVILLE

RELEASING AGENT

No Stick™

INSTRUCTIONS

No Stick™ was developed specifically to facilitate application of dental restoratives or resin cements which tend to stick to plastic and metal dental instruments. When used as directed, No Stick™ will not alter or otherwise impair the properties or performance of these materials.

DIRECTIONS FOR USE

Dispense one drop of No Stick™ on a mixing pad or dappen dish. Wet all instrument surfaces that are likely to come into contact with the material. Touch against the edge of the pad or dish to reduce excess thickness.

STORAGE

The shelf life of No Stick™ is 3 years when stored at 77° F (25° C) or below.

 Danville Materials
a Zest Anchors, LLC company
2875 Loker Avenue East
Carlsbad, CA 92010 USA
(1)760-743-7744

www.zestdent.com

ZD ZEST DENTAL
SOLUTIONS



MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany



90463 REV E

DANVILLE

RELEASING AGENT

No Stick™

INSTRUCTIONS

No Stick™ was developed specifically to facilitate application of dental restoratives or resin cements which tend to stick to plastic and metal dental instruments. When used as directed, No Stick™ will not alter or otherwise impair the properties or performance of these materials.

DIRECTIONS FOR USE

Dispense one drop of No Stick™ on a mixing pad or dappen dish. Wet all instrument surfaces that are likely to come into contact with the material. Touch against the edge of the pad or dish to reduce excess thickness.

STORAGE

The shelf life of No Stick™ is 3 years when stored at 77° F (25° C) or below.

 Danville Materials
a Zest Anchors, LLC company
2875 Loker Avenue East
Carlsbad, CA 92010 USA
(1)760-743-7744

www.zestdent.com

ZD ZEST DENTAL
SOLUTIONS



MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany



90463 REV E