

DANVILLE**LIGHT CURE DENTAL DAM****DAM
COOL****INSTRUCTIONS**

Dam Cool™ is a heatless liquid dam that has been uniquely formulated to eliminate the heat transfer problem associated with paint on dam materials. Dam Cool™ resists penetration from air abrasion particles.

DIRECTIONS FOR USE

1. Place a 20 gauge needle tip securely on the Dam Cool™ lure lock syringe.
2. Apply a coat of Dam Cool™ 1-2mm thick on the gingiva and on tooth surfaces to be protected prior to power bleaching, air abrasion, sandblasting or other procedures requiring intraoral protection, or place in areas to be blocked out prior to taking impressions. Dam Cool™ does not run so it can be placed over multiple teeth before curing.
3. Light cure for 10 seconds, depending on the light you are using.

STORAGE

The shelf life of Dam Cool™ is 3 years when stored at 77 °F (25 °C) or below.



Danville Materials
a Zest Anchors, LLC company
2875 Loker Avenue East
Carlsbad, CA 92010 USA
(1)760-743-7744

www.zestdent.com

**ZD ZEST DENTAL
SOLUTIONS**



MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany



90756 Rev E

DANVILLE**LIGHT CURE DENTAL DAM****DAM
COOL****INSTRUCTIONS**

Dam Cool™ is a heatless liquid dam that has been uniquely formulated to eliminate the heat transfer problem associated with paint on dam materials. Dam Cool™ resists penetration from air abrasion particles.

DIRECTIONS FOR USE

1. Place a 20 gauge needle tip securely on the Dam Cool™ lure lock syringe.
2. Apply a coat of Dam Cool™ 1-2mm thick on the gingiva and on tooth surfaces to be protected prior to power bleaching, air abrasion, sandblasting or other procedures requiring intraoral protection, or place in areas to be blocked out prior to taking impressions. Dam Cool™ does not run so it can be placed over multiple teeth before curing.
3. Light cure for 10 seconds, depending on the light you are using.

STORAGE

The shelf life of Dam Cool™ is 3 years when stored at 77 °F (25 °C) or below.



Danville Materials
a Zest Anchors, LLC company
2875 Loker Avenue East
Carlsbad, CA 92010 USA
(1)760-743-7744

www.zestdent.com

**ZD ZEST DENTAL
SOLUTIONS**



MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany



90756 Rev E