

No Stick

INSTRUMENT LUBE

No Stick[™] was developed specifically to facilitate application of dental restoratives or resin cements which tend to stick to plastic and metal dental instruments. When used as directed, *No Stick*[™] will not alter or otherwise impair the properties or performance of these materials.

DIRECTIONS FOR USE

Dispense one drop of *No Stick*[™] on a mixing pad or dappen dish. Wet all instrument surfaces that are likely to come into contact with the material. Touch against the edge of the pad or dish to reduce excess thickness.

STORAGE

The shelf life of *No Stick*[™] is 3 years when stored at 77°F (25°C) or below.



No Stick allows instruments to glide over composite without sticking. Does not reduce bond or composite strength.

10ml Bottle
Ref # 90461

www.danvillematerials.com

3420 Fostoria Way, Suite A-200 San Ramon, CA 94583 800.827.7940 925.973.0710 Fax 925.973.0764



Innovative Design. Reliable Performance

93310 Rev B