


PrepStart H₂O

The dust is gone! Add to any PrepStart

With Danville's PrepStart HydroAbrasion dust is captured in a concurrent water spray! The one objection to air abrasion is now gone! A cone of fine water spray surrounds the air-abrasive spray capturing virtually all dust. Water is supplied by a removable tank allowing office water to be used to be used, only an air connection is required. No water or electric connection is used, only air. Use with warm water has been reported to enhance patient comfort. Performance is not impaired. The H₂O HydroAbrasion unit is attractively styled to look attractive with the PrepStart. It is also compact and adds little weight. The HydroAbrasion system is available complete with the PrepStart, or may be ordered as an accessory to add to any existing PrepStart! The H₂O accessory comes complete attached to a new PrepStart top cover, giving a new look to your unit. Installing to a PrepStart can easily be performed with a Phillips screwdriver in about 15 minutes in your office. Installation is also available through your Dealer at Danville.

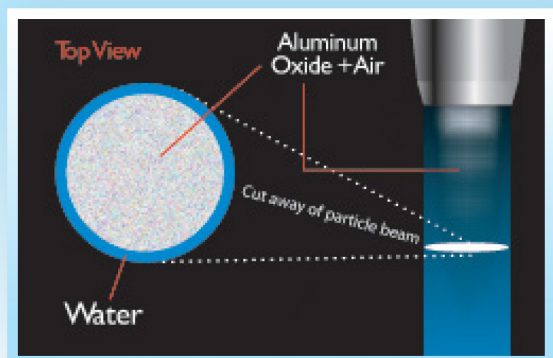

Water spray surrounds aluminum oxide to control powder spray - just like a shower curtain. Dry cutting without the mess.

FEATURES OF THE H₂O KIT:

- One, light weight, ergonomically designed handpiece
- Nozzle assemblies are detachable and autoclavable
- Nozzles are of long life carbide, nozzle body is titanium for lightest weight
- Nozzle included is popular .019" (.015" soon to be available)
- Water spray can be deactivated by a toggle switch
- Water spray automatically starts & stops with abrasive spray
- Water spray is adjustable by front knob
- Removable 250 ml, opaque water reservoir
- H₂O Kit comes attached to a new PrepStart top cover for ease of installation
- Installation can be performed in less than 15 minutes with a screwdriver

ZD ZEST DENTAL SOLUTIONS™
ZEST | DANVILLE MATERIALS | PERIOSCOPY

ZEST DENTAL SOLUTIONS
2875 LOKER AVENUE EAST
CARLSBAD, CA 92010
USA

TEL: 800.262.2310
FAX: 760.743.7975
EMAIL: SALES@ZESTDENT.COM
WWW.ZESTDENT.COM