


From the pioneers in dental air abrasion.

PrepStart™ H₂O

INSTRUCTIONS FOR RETRO-FITTING AN EXISTING PREPSTART FOOT PEDAL FOR USE WITH THE H₂O UNIT

1. Items required:
 - 1.1. Scissors
2. Slide the black cable tie away from the Red & Blue connectors approximately 6 inches.
3. Cut the clear tubing approximately 1 inch in front of the black cable tie and set aside.
4. Slide one of the white plastic tube restraints onto the clear tube and install the brass parallel tee fitting into the clear tubing, push the barb portion of the fitting all the way into the inside diameter of the tube, the parallel portion of the tee should face away from the foot pedal.
5. While holding the brass tee fitting, pull back slightly on the clear tubing with you other hand and slide the white plastic tube restraint forward until it is flush against the brass fitting.
6. Place the other white plastic tube restraint onto the clear tubing cut earlier and set aside, install the tubing onto the remaining barb of the brass tee and repeat step 5.
7. Install the black cable tie approximately a couple of inches from the blue and red connectors.
8. Your foot pedal is now ready to connect to your PrepStart H₂O unit.

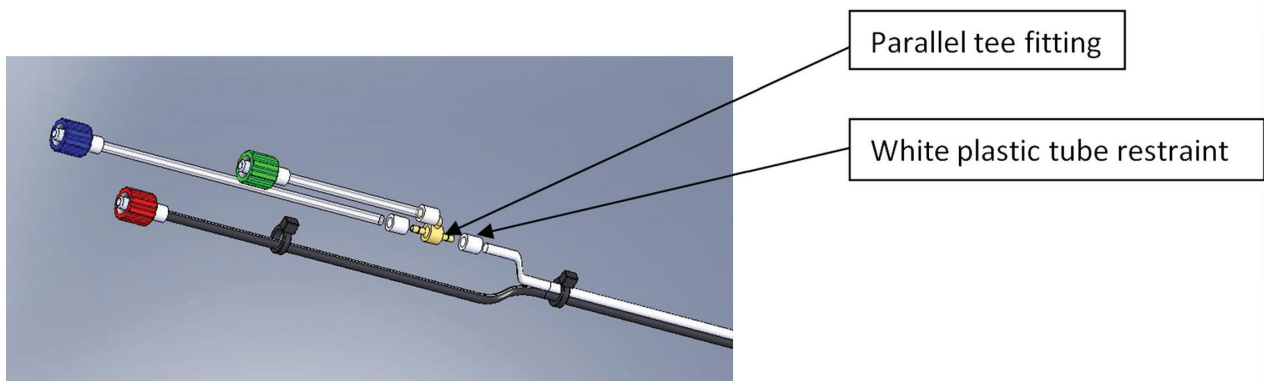

 Danville Materials
a Zest Anchors, LLC company
2875 Loker Avenue East
Carlsbad, CA 92010 USA
(1)760-743-7744

www.zestdent.com

 **ZEST DENTAL**
SOLUTIONS

 MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany



92269 REV B