StartFill 2B™ Instructions

PRODUCT DESCRIPTION

StartFill 2B™ is a dentin shading, radiopaque, fluoride releasing, dual-cured, strong, flowable, microhybrid composite. StartFill 2B is also ideal for luting posts, core buildups, crowns, veneers and any application where light transmission is in question. Being a dual-cure, StartFill 2B can be rapidly cured with a light. StartFill 2B contains 61% filler by weight (41% by volume).

INDICATIONS FOR USE

Dual-cure polymer-based dental restorative that when applied to dental surfaces pretreated with suitable primers or adhesives is indicated for use as the first increment under posterior composites; for core buildup; and for luting posts, crowns, veneers and any application where light transmission may be inadequate.

PRECAUTIONS

- Do not store the composite material in proximity to eugenol-containing products, nor let the composite come into contact with materials containing eugenol. Eugenol can impair the hardening of the composite and cause discoloration.
- Contact of resin pastes with skin should be avoided, especially by anyone having known resin allergies.
- Apply bonding agent per manufacturer’s instructions.

MIXING INSTRUCTIONS

A. Automix Syringe (one double barrel)

- Remove twist off cap and attach automix tip.
- Depress plunger with smooth, even pressure until no air is left in mix tip.
- Syringe StartFill 2B directly into restoration.
- Do not remove the automix tip after use. The use tip serves as a convenient seal until the next use.

B. Base & Catalyst Syringes (individual)

- Hand mix equal portions of the StartFill 2B Base and Catalyst.
- Place StartFill 2B using a Centrix needle tip or similar syringe tip.
- The tips and silicone caps are color coded. It is suggested that white caps be used on white tips and black caps be used on black tips to prevent cross-contamination of base and catalyst.

INSTRUCTIONS FOR USE

Directed Shrinkage Technique for Posterior Composites

StartFill 2B is ideal for the “Directed Shrinkage” technique^{1,2,3} for composite placement.

- Isolate tooth with a rubber dam.
- Cut conservative prep. Use of CariesFinder (Danville Materials), or other caries detection dye as a guide is suggested.
- Etch enamel and dentin for 15 to 30 seconds with a liquid phosphoric acid etchant such as SureEtch™ (Danville Materials).
- A highly recommended optional step to reduce sensitivity is to apply MicroPrime™ (Danville Materials) desensitizing agent for 20 seconds (minimum) penetration time. (Note: It is not recommended to MicroPrime with self etch primers.)
- Apply bonding agent and cure.
- Apply StartFill 2B and allow to self cure for at least 2 1/2 minutes. (To achieve directed shrinkage effect, the minimum depth is a Dycal-like layer which includes the gingival margin.) (Note: StartFill 2B can be light cured for faster set but that cure negates the directed shrinkage effect.)
- Place a light-cure posterior composite and cure. (There is no need for bonding agent between the StartFill 2B and the overlying composite).
- OR—*
- A bite form technique, using a sectional matrix system such as Contact Matrix™ (Danville Materials), may be used to form the composite. Cure the composite, remove the Contact Ring, apply glycerine gel on the occlusal and have the patient bite into centric occlusion. Cure the composite while the patient’s mouth is closed. Have the patient open and cure again.
- Remove the matrix.
- Finish and polish.
- Avoid cross-contamination between patients by replacing mixing tip between patients. Disposable handpiece or syringe plastic barrier sleeves may provide greater prevention of cross contamination. Use StartFill 2B automix syringe with new tip into barrier sleeve, piercing only the automix tip through the plastic. If used with Insert tip, follow Centrix directions for use.

STORAGE

Refrigerate for full shelf life. Use at room temperature.

DEFINITIONS OF SYMBOLS

The following symbols may appear on the product packaging or labeling.

Symbol	Definition	Symbol	Definition
REF	Reference number	LOT	Lot number
	Use by		Upper limit of temperature
RxOnly	Caution: Federal law restricts this device to sale by or on the order of a dentist.	EC REP	European Community Authorized Representative
	Manufacturer	CE 0086	European Mark of Conformity
	Keep away from sunlight		Consult instructions for use

- Bertolotti R.L. *PP&A Aesthetic Chronicle* 1991; 3, 53-58.
- Ferrari M. *PP&A Aesthetic Chronicle* 1993; 5, 29-36.
- Garberoglio R. et al. *Am J Dent* 1995; 8, 303-307.

 Danville Materials 2875 Loker Avenue East Carlsbad, CA 92010 USA (714)670-743-7744 www.zestdent.com	 MDSS GmbH Schiffgraben 41 30175 Hannover Germany	
RxOnly		90393 Rev L

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