



CRITERIA FOR STARTING AN ENCOMPASS CASE

1. Patient must be fully edentulous
2. Patient must have a well fitting denture
 - * *Either hard relined or reline impression inside denture*
3. Include digital opposing & bite

REQUIRED ITEMS

1. Complete digital impression scan of denture with radiographic markers in place
 - * *Please be sure to include the full palate of the denture as well as the intaglio surface and flanges*
2. CBCT of patient wearing denture with radiographic markers (out of occlusion) biting on cotton rolls
3. Patient photos with denture in:
 - * *Full Face Smiling*
 - * *High Lip Line*
 - * *Low Lip Line*
 - * *Profile*
4. Completed digital Rx form sent through NavaGation.net

FOR MORE DETAILED INFO SCAN QR CODE FOR VIDEOS

1. What We Need to Start a Case
2. Patient Selection
3. How to Scan Your Patient
4. What You Get in the ENCOMPASS Box



TIPS FOR SUBMITTING A SUCCESSFUL CASE

1. Please make sure that your patient has a well fitting denture
 - * If a reline is needed, chairside reline is acceptable
 - * *Refrain from soft relines*
 - * Impression material is acceptable
 - * *Be sure to trim away any excess*
 - Note:* A loose denture will result in a loose guide
2. Place your radiographic markers, evenly spaced across the buccal surface
 - * Typically in the 3, 6, 11, & 14 general regions
 - * You will also need to place 2 radiographic markers in the palatal surface (Not the intaglio surface of the denture)
 - Note:* We recommend that you use either VF-20 or CT spot radiographic markers
3. Place the denture in your hand and digitize the denture completely, (capturing the intaglio surface)
 - * Please make sure that the radiographic markers are visible in the scan
 - * Continue scanning opposing, then place the denture back into the mouth
 - * Scan the bite
4. Perform a CBCT scan of the patient (out of occlusion) while biting on cotton rolls, to ensure the denture stays fully seated
 - * Please be sure that the radiographic markers stay in place, as movement may result in restarting the process
5. *Once you have completed these steps, you can upload your case through the ZEST portal on our website NavaGation.net.*
 - * *Remember to upload the patient smile and profile photos*