Instructions for Use
Zest CHAISIDE® Denture Prep and Polish Kit
IMPORTANT: This document contains the most current instructions for Use. Please read and retain.

DESCRIPTION
The Zest CHAISIDE Denture Prep and Polish Kit contains all the necessary instruments to prepare and finish dentures over a pick-up of a LOCATOR® or SATURN® attachment. The system includes a recess burs, undercar bur, wire bur, hole bur, trim bur, grind bur, and polisher. **INDICATIONS**

The Zest CHAISIDE Denture Prep and Polish Kit is intended for use with partial and acrylic dentures as well as a partial or chairside laboratory procedure.

CARTRIDGE
Federal (USA) law restricts this device to sale by or on the order of a dentist.

STORAGE AND HANDLING

The Zest CHAISIDE Prep and Polish Kit in its undamaged, original packaging is not subject to any special considerations for storage or handling (during transport and storage).

WARNINGS, PRECAUTIONS & DIRECTIONS FOR USE

* Zest CHAISIDE Prep and Polish Kit is not affected by the MR environment and therefore has not been tested for heating or migration in the MR environment.

* The use of this or any dental restorative product requires that the clinician and other specialists be thoroughly familiar with the application of these products. This document also provides information on the instruments and kit sterilization requirements.

* Care should be taken in choosing the instruments to be used in the denture during use to avoid damage to the instrument or piece of work due to excessive heat generation.

* Improper use of instruments may contribute to material damage, which may lead to poor results and pose a danger to the operator.

* The Polisher is for single use only in cases where restorative materials are being used.

* The Kit and individual instruments are packaged, sterilized and pre-sterilized by instructions prior to use.

Refer to and strictly adhere to the manufacturers' instructions for use for the reaction time and suitability of disinfectants and detergents.

Preparation

* All instruments need to be removed from the products directly after use (within two hours maximum).

To do so, use running water or an FDA approved and/or CE Marked disinfection solution.

The disinfectant solution should be established, suitable, and compatible with the instruments.

CLEANING INSTRUCTIONS

* Place the instruments in the cleaning bath for the prescribed contact time and concentration, take care that the instruments do not come into contact with other.

* Do not use any container of the instrument, it is recommended to use an ultrasonic bath for cleaning.

* The Block must be cleaned and disinfected without instruments loaded.

* The Block is not suitable for cleaning using a thermal disinfector (disinfect RODG).

* Remove the instruments from the cleaning solution and rinse for at least 3 minutes with tap water.

* Remove excess moisture from the components with a clean, absorbent, and non-sudging wipe.

RECOMMENDED STERILIZATION INSTRUCTIONS

All instruments and instruments are supplied

* Check instruments for damaged surfaces, chipped off edges, cracks or any other damaged instrument. Products that are damaged or damaged must be cleaned and disinfected until all visible contaminants have been removed.

* Attach the instruments to the sterilization block in the dedicated bur block. Wrap the bur block using disposable sterilization packaging and/or sterilization containers. Sterile assembled bur block according to the parameters listed in the following table:

	Gravity	Autoclave	Pre-Vacuum	Autoclave
Temperature	132°C	132°C		
Exposure Time	25 minutes		4 minutes	
Dry Time	30 minutes		20 minutes	

STERILE PREPARATION PROCEDURES

The steps below are guidelines to help process the dentures for implant-retained restorations:

1. Use the Recur to create a void in the denture. Use the fresa to create a recess for the LOCATOR or SATURN cap.

2. It is recommended that you use the Undercut bur to create an undercut on the LOCATOR or SATURN cap to enable better mechanical retention of the pick-up material to the prosthesis. The Undercut bur can also be used to create additional clearance in the recess if needed.

3. Use the Vent Bur to create a small hole through the denture recess, which enables the visibility of the LOCATOR or SATURN cap and allows for excess pick-up material to flow out.

4. Use the Trim Bur to carefully trim the flanges and any edges on the LOCATOR or SATURN cap.

5. Use the Grind Bur carefully trim around the attachment pick-up material, if needed.

6. Use the Polisher on all rough edges to create a smooth denture surface.

Recommended Speeds:

Zest Code Application Recommended speed rpm Maximum speed rpm

09577 Undercut 20,000 50,000

09578 Vent 18,000 50,000

09579 Trim 20,000 35,000

09576 Recess 18,000 30,000

09583 Grind 20,000 35,000

09580 Polish 7,000 12,000

No coolant is needed during use.

Explanation of Symbols on Outer Packaging Label

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