




CHAIRSIDE®

Attachment Processing Material

THE CHAIRSIDE ATTACHMENT PROCESSING MATERIAL
YOU HAVE BEEN WAITING FOR.



When You Have To Do A Pick-Up – Pick ZEST!

 **CHAIRSIDE**® is a unique **Attachment Processing Material** from Zest Dental Solutions, the manufacturer of the trusted LOCATOR® Abutments, LOCATOR Overdenture Implant (LODI) System and SATURNO® Narrow Diameter Implant System (SNDI). CHAIRSIDE® Attachment Processing Material was designed for ease of use and predictability when processing attachments into full and partial overdentures, using either a chairside or laboratory procedure.

Clinician input contributed to a formulation that has the most sought after handling characteristics, which requires no primer, and is self-curing — all at a reduced cost per case*. With all of these benefits, why would you choose a processing material from any other company than the leading manufacturer of overdenture products for more than 40 years — **Zest Dental Solutions**.

CHAIRSIDE Attachment Processing Material Key Attributes:

Cost Efficient

- No primer required for reduced chair time
- Shorter mixing tip reduces material waste and cost per case*
- Two size options to suit your implant practice
 - 4mL syringe for pickup of LOCATOR and SATURNO Denture Attachment Housings
 - Large 18mL cartridge ideal for full-arch fixed provisional prostheses utilizing immediate load protocols such as DIEM® 2 or All-On-4®

Simple Application

- Ideal viscosity for accurate application
- Bonds to itself for efficient void fill and touch-ups
- Dual cure capability provides convenient self or light cure flexibility

Patient Friendly


- Low curing temperature ensures patient comfort
- Odorless and tasteless to increase patient acceptance

"CHAIRSIDE Attachment Processing Material by Zest is a game-changer!" said Dr. Michael Scherer. "Zest has developed a material that is perfect for processing attachments...It's so easy to use, no messy primer is required, and the material offers dual-curing flexibility (self or light), thereby making the time I spend connecting the denture caps to the overdenture highly efficient and effective."


*Michael Scherer, DMD, MS, FACP
Sonora, California USA*

*Compared to the leading brands.


Six Simple Steps For Chairside Processing LOCATOR R-Tx® Attachments Into Overdentures

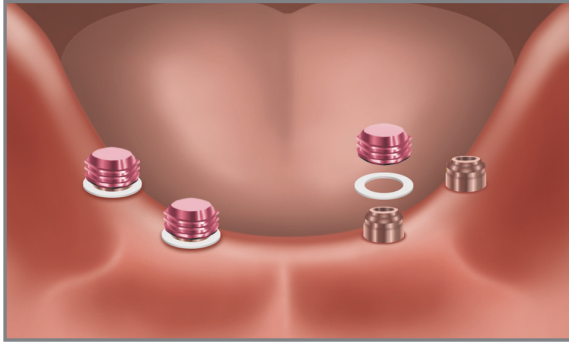


1. Place a White Block Out Spacer Ring around each Abutment and press it down to the tissue. Place a Denture Attachment Housing with a Black Processing Insert onto each Abutment and press down firmly to ensure engagement of the Abutment.

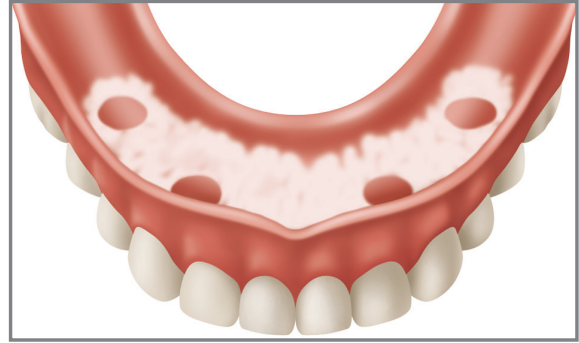


2. Apply fit check marking paste inside the denture. Insert it into the mouth over the Denture Attachment Housings. Mark the areas where the denture will need to be relieved to allow space for the Housings to be picked up.

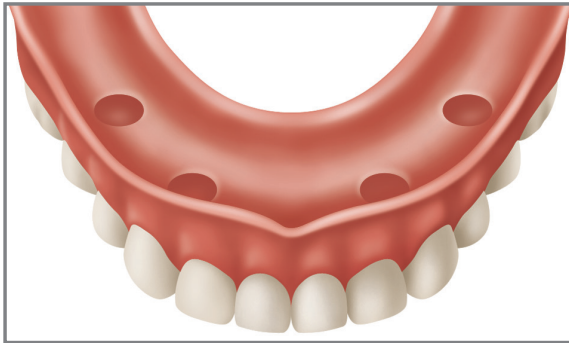


3. Relieve the areas marked with an acrylic bur. Try in the denture to verify that the Denture Attachment Housings are not in contact with the acrylic in any area. Cut lingual/palatal vent windows in the denture to visualize full seating and for excess material to vent. Cut an undercut below the intaglio surface of each Denture Attachment Housing relief area for mechanical retention.

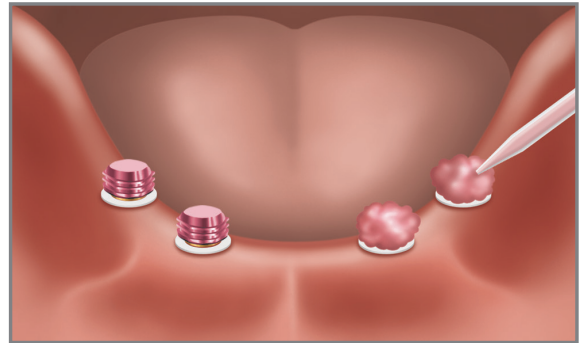

4. Dry the Denture Attachment Housings. Apply a small amount of CHAIRSIDE® Material around the circumference of each Housing. Place CHAIRSIDE Material into the relief areas of the denture and seat it over the Housings and onto the tissue. Have the patient close into light occlusion and hold while the CHAIRSIDE Material sets. The set time of the material from the time the material is mixed to the time it is set is five (5) minutes. The working time for the material is one (1) minute, forty-five (45) seconds.

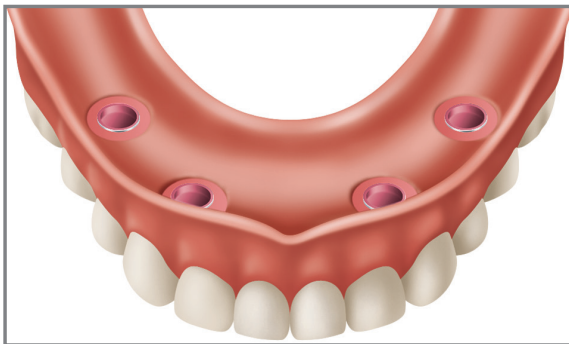

5. Disengage the denture from the Abutments and remove from the mouth. Verify that the Denture Attachment Housings have been securely processed into the denture. Fill any voids, light cure and polish the denture. The material will bond to itself and will cure within 30 seconds with light application.

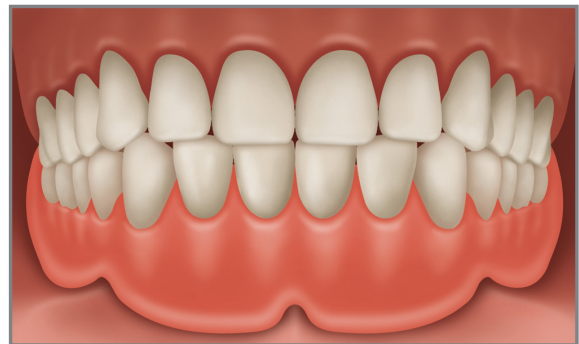

6. Place the denture in the mouth and press down to engage the Inserts on the Attachment System.



CHAIRSIDE®

Attachment Processing Material

PRODUCT ORDERING INFORMATION

ITEM NUMBER	DESCRIPTION	PRICE
09566	CHAIRSIDE® Attachment Processing Material 4mL Syringe (15 brown mixing tips and 15 angled tips included in each package)	\$87.50
09565	CHAIRSIDE Attachment Processing Material, 18mL Cartridge (10 blue mixing tips included in each package)	\$252.00
09125-15	CHAIRSIDE Attachment Processing Material Short Mixing Tips (15) (to use with 4mL syringe)	\$17.00



4mL Syringe and Plunger



18mL Cartridge



Mixing Tips (15)



Angled Tips (15)



Mixing Tips (10)

Try the **CHAIRSIDE Attachment Processing Material** you have been waiting for today
by calling 800.262.2310 or visiting www.zestdent.com

ZD ZEST DENTAL SOLUTIONS
ZEST | DANVILLE MATERIALS | PERIOSCOPY

ZEST DENTAL SOLUTIONS
2875 LOKER AVENUE EAST
CARLSBAD, CA 92010
USA

TEL: 800.262.2310
FAX: 760.743.7975
EMAIL: SALES@ZESTDENT.COM
WWW.ZESTDENT.COM

DISTRIBUTED BY: