

CONVERT WITH CONFIDENCE

Transition from a screw-retained fixed prosthesis to a removable overdenture solution with ease by utilizing the existing implant manufactured multi-unit abutments with the LOCATOR® Attachment System for Multi-Unit Abutments.




Screw-Retained Solution with existing Implant Manufacturer Multi-Unit Abutments



Removable overdenture solution with the LOCATOR Attachment System for Multi-Unit Abutments


†Case images courtesy of Dr. Michael Steffen, Tulsa, OK.

LOCATOR® ATTACHMENT SYSTEM FOR MULTI-UNIT ABUTMENTS

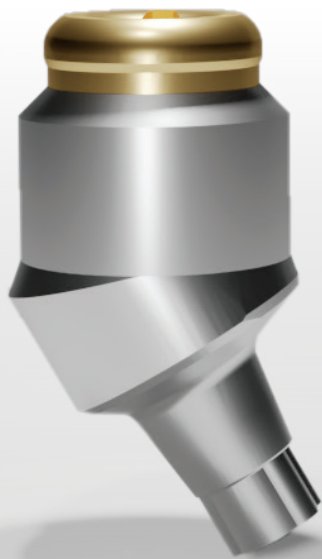


- Self-aligning, dual retentive design to enhance engagement and stability of the overdenture
- Low profile to maximize interocclusal space
- Customizable levels of retention

SIMPLE RESTORATIVE STEPS

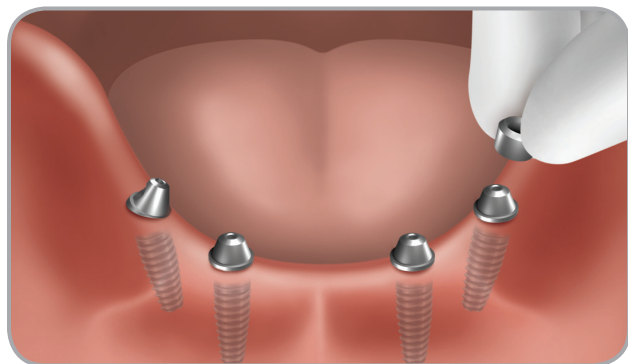


Torque multi-unit abutments to manufacturer's recommended torque value and place LOCATOR Titanium Collar on each multi-unit abutment.

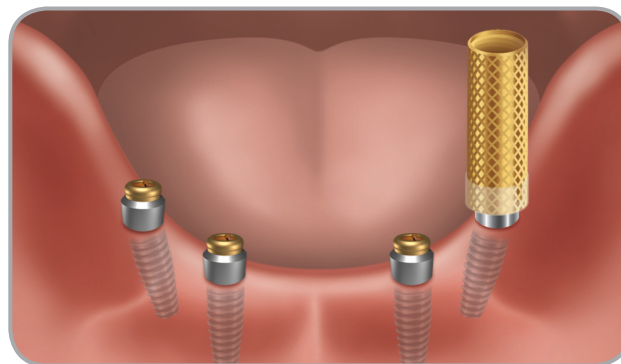

Thread LOCATOR Abutments to multi-unit abutments and torque to 20 Ncm with LOCATOR Torque Wrench Driver. Proceed with the standard procedure of the LOCATOR Attachment System.

Warning: The use of torque values higher than 20 Ncm could potentially fracture the Multi-Unit LOCATOR Abutment.

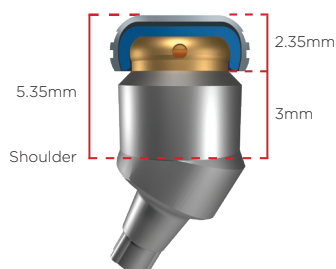

Note: 5.35mm of vertical clearance is required from the shoulder of the multi-unit abutment.

Designed for free-standing and castable bar-splinted applications.

Multi-Unit Abutment
w/ Titanium Collar
08909



Male Processing Pkg
08519



Extended Range
Male Processing Pkg
08540



Abutment
w/Delrin Collar
08917



Bar Male
Processing Pkg
08028



Part Number	Description
Free-Standing Application	
08909-2	LOCATOR Abutment and Collar for Multi-Unit (2 Pack or 10 Pack)
08909-10	
08519-2	LOCATOR Male Processing Package (Includes Denture Cap with Black Processing Male, Standard Range Males: Clear, Pink, Blue and Block-Out Spacer) (2 Pack or 10 Pack)
08519-10	
8540-2	LOCATOR Extended Range Male Processing Package (Includes Denture Cap with Black Processing Male, Extended Range Males: Green, Orange, Red and Block-Out Spacer) (2 Pack or 10 Pack)
08540-10	
Bar Splinted Application	
08917-2	LOCATOR Abutment and Castable Collar for Multi-Unit (2 Pack or 10 Pack)
08917-10	
08028-2	LOCATOR Bar Male Processing Package (Includes Denture Cap with Yellow Processing Male, Standard Range Males: Clear, Pink, Blue and Block-Out Spacer)
08028-10	

Call ZEST Customer Service for a complete list of multi-unit abutment manufacturers compatible with the LOCATOR Attachment System for Multi-Unit Abutments. For restorative steps on the Bar Splinted Application, refer to the LOCATOR Attachment System Technique Manual which can be found at www.zestdent.com. Please refer to the Instructions for Use for further information.