Bulk EZ® Dual-Cure Bulk-Fill Composite with IntelliTek® Technology

INSTRUCTIONS FOR USE

PRODUCT DESCRIPTION

Bulk EZ® is an easy-to-place, dual-cure, bulk-fill composite that combines flowable cavity adaptation with high strength and wear resistance—all in one simple step. The self-cure IntelliTek® Technology is designed to specifically control and direct shrinkage for the purpose of eliminating microleakage in posterior restorations. Additionally, Bulk EZ is highly color-stable over time and compatible with a wide variety of bonding agents. Bulk EZ is designed to offer an unlimited depth of cure in a single application without multiple layering and light-curing when allowed to self-cure.

INDICATIONS FOR USE

Bulk EZ is a dual-cure (auto-cure with light-cure acceleration) resin-based dental restorative that when applied to dental surfaces prepared with suitable primers or adhesives is indicated for use as:

- Direct restoration, Ideally Class I and II
- Cavity liner
- Core build-up

WARNINGS AND PRECAUTIONS

1. Uncured resin material can cause irritation, especially in people known to be allergic to methacrylates. Wash hands after handling material.

2. The composite is formulated to be used at room temperature. Shelf life at room temperature is stated on the syringe.

3. Do not store the composite material in proximity of eugenol-containing products. Do not let the composite come into contact with material containing eugenol. Eugenol can impair the hardening of the composite and cause discoloration.

4. Do not expose materials to elevated temperatures or intense light.

5. Avoid cross-contamination by using a new mixing tip for every patient and using a barrier sleeve.

APPLICATION:

1. Isolate tooth to prevent contamination of blood and/or saliva. Suggested uses include a rubber dam, Isolite® System, or Zest's Dam Cool™ light-cured dental dam.

2. Complete conservative cavity preparation with conventional means or with a hydro-abrasive system such as Zest's PrepStart™ H2O.

3. Use of Zest's Caries Finder™ is recommended to accurately visualize caries removal without unnecessary tooth reduction.

4. Place sectional matrix such as Zest's Mega V™ Ring contact matrix system to obtain natural interproximal contour as needed. Ensure good adaptation to minimize clean-up.

5. Apply etch-and-rinse such as Zest's Sure Etch™ per manufacturer's instructions.

6. Apply bonding agent such as Zest's Prelude™ or Prelude One™ per manufacturer's instructions.

Note: Bulk EZ is compatible with a wide variety of methacrylate-based dental bonding agents/systems as well as etching techniques.

MIXING AND FILLING:

1. Remove the twist-off cap.
2. Before each use, blend material until both base and catalyst are extruded out.
3. Manually blend the metal cannula on the mixing tip to the desired angle.
4. Securely attach a mixing tip by twisting it onto the head of the syringe, bleed a small amount of material to ensure even mixing. To avoid cross-contamination, use a new mixing tip for every patient.
5. To avoid cross-contamination, use a barrier sleeve to cover the syringe.

Note: Avoid strong direct light on the transparent mixing chamber as it could speed up the thickening process.

CAVITY FILLING WITHOUT CAPPING:

1. Immediately place the mixing tip at the deepest part of the preparation and fill while keeping tip within the material to avoid introducing air. End filling 0.5-1.0 mm from the occlusal surface, or half of the top enamel layer.
2. Allow material to self-cure for at least 90 seconds. Subsequent light-curing of the top surface for 10 seconds is optional.
3. Proceed to finishing and polishing steps.

FOR CAVITY FILLING WITH CAPPING:

1. Immediately place the mixing tip at the deepest part of the preparation and fill while keeping tip within the material to avoid introducing air. End filling 0.5-1.0 mm from the occlusal surface, or half of the top enamel layer.
2. Place a universal composite and sculpt it to the desired contour. Cure the composite according to manufacturer's instructions.

Note: It is recommended to leave the Bulk EZ mixing tip on the syringe as a cap until the next application.

FINISHING AND POLISHING

1. Complete restoration with preferred finishing and polishing steps.
2. Polish to a high gloss with disc or composite polishing points. Interproximal finishing/polishing is accomplished with fine grit finishing/polishing strips.

STORAGE

Store at or below 77°F (25°C). Use at room temperature.

DEFINITION OF SYMBOLS

The following symbols may appear on the product packaging or labeling.

| SYMBOL | TITLE | EXPLANATORY TEXT | STANDARD | REFERENCE |
|--------|--|--|--|--------------------|
| | Manufacturer | Indicates the medical device manufacturer, as defined in EU Directive 93/42/EEC | ISO 15223-1 | 5.1.1 |
| | European Community Authorized Representative | Indicates the authorized representative in the European Community | ISO 15223-1 | 5.1.2 |
| | Catalogue Number | Indicates the manufacturer's catalogue number so that the medical device can be identified | ISO 15223-1 | 5.1.6 |
| | Batch Code | Indicates the manufacturer's batch code so that the batch or lot can be identified | ISO 15223-1 | 5.1.5 |
| | Consult Instructions for Use | Indicates the need for the user to consult the instructions for use | ISO 15223-1 | 5.4.3 |
| | Use-by date | Indicates the date after which the medical device is not to be used | ISO 15223-1 | 5.1.4 |
| | Upper Limit of Temperature | Indicates the temperature limits to which the medical device can be safely exposed | ISO 15223-1 | 5.3.6 |
| | European Mark of Conformity | Indicates device is in conformance with Medical Device Directive 93/42/EEC | MDD 93/42/EEC | Annex XII |
| | Rx Only | Caution: Federal law restricts this device to sale or on the order of a dentist | US Code of Federal Regulations, Title 21 | 801.15(c)(1)(i)(F) |
| | Keep away from sunlight | Indicates a medical device that needs protection from light sources | ISO 15223-1 | 5.3.2 |

Prestige is not approved by Health Canada.

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