



CHAIRSIDE®

Attachment Processing Material

THE BEST PICK-UP MATERIAL FROM THE #1 LEADER IN OVERDENTURE ATTACHMENTS.


CHAIRSIDE® is a unique **Attachment Processing Material** from Zest Dental Solutions, the manufacturer of the market-leading LOCATOR® overdenture products for more than 40 years. It was designed for ease of use and predictability when processing attachments into full and partial overdentures, using either a chairside or laboratory procedure.

The dedicated formulations allow clinicians to carry out their procedures the way they want to, easily, quickly, and at a reduced cost per case



Six Simple Steps For Chairside Processing Attachments Into Overdentures

1

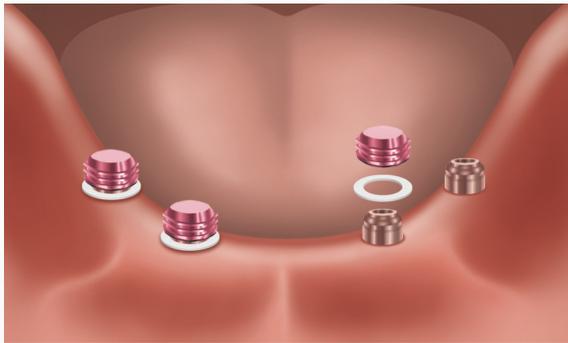

Place a White Block Out Spacer Ring around each Abutment and press it down to the tissue. Place a Denture Attachment Housing with a Black Processing Insert onto each Abutment and press down firmly to ensure engagement of the Abutment.

2




Apply fit check marking paste inside the denture. Insert it into the mouth over the Denture Attachment Housings. Mark the areas where the denture will need to be relieved to allow space for the Housings to be picked up.

3




Relieve the areas marked with an acrylic bur. Try in the denture to verify that the Denture Attachment Housings are not in contact with the acrylic in any area. Cut lingual/palatal vent windows in the denture to visualize full seating and for excess material to vent. Cut an undercut below the intaglio surface of each Denture Attachment Housing relief area for mechanical retention.

4

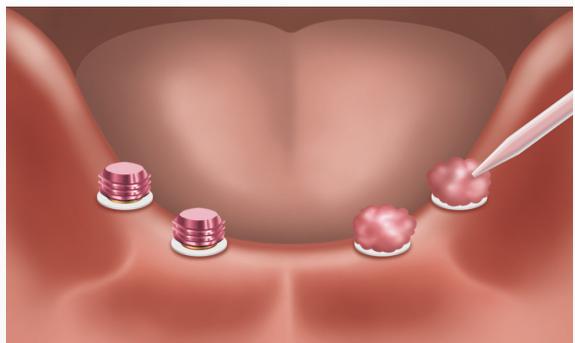

Dry the Denture Attachment Housings. Apply a small amount of CHAIRSIDE® Material around the circumference of each Housing. Place CHAIRSIDE Material into the relief areas of the denture and seat it over the Housings and onto the tissue. Have the patient close into light occlusion and hold while the CHAIRSIDE Material sets. The work time and set time of the material varies between Normal Set and Fast Set varieties.

5

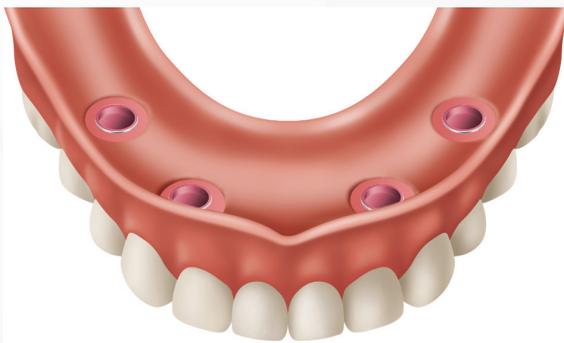

Disengage the denture from the Abutments and remove from the mouth. Verify that the Denture Attachment Housings have been securely processed into the denture. Fill any voids, light cure and polish the denture. The material will bond to itself and will cure within 30 seconds with light application.

6



Place the denture in the mouth and press down to engage the Inserts on the Attachment System.