

These instructions are to ASSIST you in the Iveri Whitening process. As a Dental Professional use your discretion as each individual client is different and may require additional treatment.

Client should read and sign Iveri Whitening Consent Form. For Optimum Results a Professional Cleaning is Recommended.



STEP 1

Application of Liquid Dental Dam Use Cheek Retractor to Retract Cheeks Dry Gums using Gauze or Air Water Syringe Apply Liquid Dam around Margins of the gums for Mandible & Maxilla

Light Cure using Blue LED Light for 5-10 Seconds



STEP 2

Insertion of Gel-Lined MouthTray Advise Client to Bite Gentle into the Tray Gel can be Maneuvered, if patients teeth are tilted or rotated, by applying Lite pressure to the tray with your fingers.



STEP 3

LED Light
LED Light is placed in front of Gel Lined Tray for 15-20 minutes



STEP 4

End of Procedure Gel Lined tray is removed Client can Rinse with water For Additional Session(s) do not remove liquid dental dam from gums. Repeat steps 2-4 otherwise dental dam can be removed

Office: 416.231.9444 Toll Free: 1.888.451.0555 Fax: 1.888.522.7996

info@iveriwhitening.com www.iveriwhitening.com