IVERI REGULAR STRENGTH



These instructions are to ASSIST you in the Iveri Whitening process. As a Dental Professional use your discretion as each individual client is different and may require additional treatment.

Client should read and sign Iveri Whitening Consent Form. For Optimum Results a Professional Cleaning is Recommended.



STEP 1

Application of Vitamin E Oil Break Vitamin E Oil Applicator at Red Line Apply Vitamin E Oil to Gingiva



STEP 2

Insertion of Gel-Lined Mouth Tray Advise Client to Bite Gentle into Whitening Tray Gel can be Maneuvered, if Clients teeth are tilted or rotated, by applying Lite pressure to the tray with your fingers



STEP 3

LED Light
LED Light is placed in front of Gel Lined Tray
for 15-20 minutes



STEP 4

End of Procedure Gel Lined tray is removed Client can Rinse with water

Optional Step - Application of Liquid Dental Dam May be Recommended for Multiple Back to Back Sessions.

Office: 416.231.9444
Toll Free: 1.888.451.0555

info@iveriwhitening.com www.iveriwhitening.com