INSTRUCTIONS

INGREDIENTS

Glycerin, Urea Carbamide Peroxide (35%), Propylene Glycol, Water, Carbomer, Potassium Nitrate, Triethanolamine, Sodium Saccharin, Potassium Sorbate, Flavor (aroma)

CAUTION

For cosmetic use only. Not recommended if patient is pregnant, nursing or is under 14 years of age. Do not use if patient presents allergies to any of the ingredients. If in contact with eyes, rinse immediately. Store in a cool place.

Multiple treatments daily may result in sensitivity to your teeth or gums. If you experience sensitivity, refrain from use for one day. If sensitivity persists, consult your dentist.

INSTRUCTIONS

Open the bottom battery cover of the Whitening LED Light and remove the plastic covering from the battery (an additional battery is provided for future use).

Connect the whitening tray to the LED Light by inserting one edge of the light into the tray and pull the other edge over.

Dispense gel in a thin layer along the front edges of the tray (do this for both arches).

Note: Do not over fill the tray. Turn the Light on and insert tray into your mouth. Move the tray side to side to ensure the gel covers all tooth surfaces.

Note: The recommended treatment time is 20 minutes, however, the LED Light will automatically shut off every 10 minutes.

After treatment, remove the tray, rinse your mouth with warm water, and brush your teeth to remove any excess gel.

For additional results, treatment can be repeated immediately by refilling the tray with more gel.

MAINTENANCE

This kit provides anywhere from 8-12 dual arch treatments, based on how much gel is extruded. Only apply a small amount to the front of the tray. Continue treatment until the desired shade is achieved. Contact your dentist for additional gel, as needed.

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