

Saliva on molars a problem?

H₂Ortho=Bond™

Self-Etch Orthodontic Adhesive System

- Quick & Easy Application
- Metal/Ceramic Brackets, Lingual Retainers, Invisalign Attachments
- Clear Occlusal Sealant - Water/Saliva Compatible

Saliva on 2nd Molars is no longer a problem when bonding brackets! Solvent-free H₂Ortho=Bond is compatible with water or saliva!

This fast, reliable light cured adhesive works with light-cured, auto-cured and dual-cured composite cements.

EXCELLENT FOR:

- Metal/ceramic brackets, lingual retainers, Invisalign attachments.
- Making a clear occlusal sealant—no need to worry about water saliva in grooves/fissures.



H2Ortho=Bond
(1) 5ml bottle
Ref # 93139

Application is quick and easy:

- 1** Clean surface with oil-free pumice on a prophy cup or SA85 powder in a MicroEtcher
- 2** On the wet (H₂O or saliva) tooth surface, scrub the H₂Ortho=Bond on with a microfiber brush tip; Move it around for 10 seconds
- 3** Move the material around with a gentle air stream for 10 seconds, then increase to a heavier air stream for 10 seconds
- 4** Light cure the adhesive for 10 seconds
- 5** Place attachment or brackets on bonded surface with ortho cement, position, and light cure

ZD ZEST DENTAL SOLUTIONS®
ZEST | DANVILLE MATERIALS | PERIOSCOPY

ZEST DENTAL SOLUTIONS
2875 LOKER AVENUE EAST
CARLSBAD, CA 92010
USA

TEL: 800.262.2310
FAX: 760.743.7975
EMAIL: SALES@ZESTDENT.COM
WWW.ZESTDENT.COM