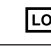
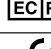
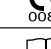
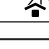
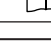
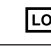
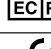
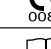
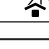
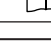
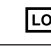
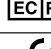
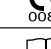
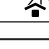
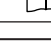
<p><b>Prelude™</b>  <b>Instructions for Use</b></p> <p><b>Product Description</b></p> Prelude™ is a nanohybrid, light cured, self-etch or total-etch dental adhesive ideal for bonding to dentin, enamel, porcelain and metal. Its low film thickness at 5 microns does not interfere with the fit of indirect restorations. <p><b>Indications for Use</b></p> Prelude is a light cured adhesive designed for direct restorations, i.e. composites and compomers to enamel and/or dentin, composite repairs, porcelain repairs and post and core build-up materials, and indirect restorations such as veneers, onlays or inlays, crowns and bridges. <p><b>Precautions</b></p> <ul style="list-style-type: none"><li>Prelude Primer, Adhesive and Dual/Self-Cure Link are never mixed.</li> <li><b>Eugenol</b> will inhibit set. Phosphoric etchant will remove residual eugenol.</li> <li><b>Avoid saliva and blood contamination.</b></li> <li><b>Adhesion to self-cure/dual-cure</b> composites requires Prelude Dual/Self-Cure Link. When Prelude Adhesive and Dual/Self-Cure Link are used with dual/self-cured composites Prelude will polymerize without light making it self-cure.</li> <li><b>Store &lt;85°F</b> and out of sunlight. Refrigeration may extend shelf life.</li> <li><b>Caution:</b> Unpolymerized monomers may cause skin irritation. Wash thoroughly with soap and water in case of contact.</li></ul> <p><b>Total-Etch</b></p> <ol style="list-style-type: none"><li><b>Etch tooth surface for 10 seconds</b> with phosphoric acid etchant. <b>Do not scrub. Rinse well. See below for preparation of other bonding surfaces.</b></li> <li><b>Do not blow dry. Blot excess water</b> with a new MicroBrush™, foam pellet or large paper point. <ul style="list-style-type: none"><li>Continue at Step 3.</li></ul></li></ol> <p>— <i>OR</i> —</p> <p><b>Self-Etch</b> (no etchant required)</p> <ol style="list-style-type: none"><li><b>Apply Primer</b> to damp or dry enamel and dentin. Scrub for 10 seconds with a MicroBrush™. Do not rinse.</li> <li>Use an air syringe to gently evaporate the solvent. Surface need not remain shiny. <ul style="list-style-type: none"><li>Continue at Step 3.</li></ul></li></ol> <p><b>Preparation of Other Surfaces</b></p> <p>First, air abrade all bonding surfaces of restorations being cemented or repaired using Danville's MicroEtcher™ and 50 micron Aluminum Oxide. Next, follow instruction related to surface type: Composites, Porcelain and Lithium Disilicate - Apply 5-Bond™ according to its instructions and continue at Step 3.</p> <p>Zirconia, Alumina, and Metals - Apply Z-Bond™ according to its instructions. Surface is ready for composite or cement.</p> <p>Amalgam - Begin at Self-Etch Step 1, above.</p> <p>Precious and Non-Precious Metals - Begin at Step 3. Tin-Plating of precious metal bonding surfaces improves bond strength.</p> <p><b>Adhesive Application (Follow Steps 1 and 2 and continue at Step 3 below.):</b></p> <ol style="list-style-type: none"><li><b>Apply Adhesive</b> with a new MicroBrush™. <b>Scrub for 10 seconds.</b></li> <li><b>Evaporate solvent</b> with gentle air stream. Increase air pressure to <b>thin Adhesive.</b></li> <li><b>Light Cure 10 seconds.</b> Light cure is optional when using Dual/Self-Cure Link with self-cure and dual-cure composites.</li></ol> <p><b>For direct restorations:</b> Place composite (flowable preferred) or compomer restorative and light cure.</p> <p><b>For indirect restorations:</b> See "Preparation of Other Surfaces." Follow instructions below for using Dual/Self-Cure Link on every surface where Adhesive has been applied.</p>	<p>ENGLISH</p> <p><i>Upozorenje: Dual/Self-Cure Link može štraktiti uporabni rok nekih samopolimerizirajućih i dvojno polimerizirajućih kompozita.</i></p> <p><b>Definicija simbola</b></p> <p>Sljedeći se simboli mogu pojavljivati na pakiranju ili etiketi proizvoda.</p> <table> <tbody><tr> <td><b>Simbol</b></td> <td><b>Definicija</b></td> <td><b>Simbol</b></td> <td><b>Definicija</b></td></tr> <tr> <td></td> <td>Referentni broj</td> <td></td> <td>Broj serije</td></tr> <tr> <td></td> <td>Upotrijebiti do</td> <td></td> <td>Gornja granica temperature</td></tr> <tr> <td><b>RxOnly</b></td> <td>Oprez: Savezni zakon ograničava prodaju ovog uređaja na stomatologa ili po nalogu stomatologa.</td> <td></td> <td>Ovlašteni predstavnik za Europsku zajednicu</td></tr> <tr> <td></td> <td>Proizvođač</td> <td></td> <td>Europska oznaka sukladnosti</td></tr> <tr> <td></td> <td>Držati podalje od sunčeve svjetlosti</td> <td></td> <td>Pogledajte upute za uporabu</td></tr> </tbody></table>	<b>Simbol</b>	<b>Definicija</b>	<b>Simbol</b>	<b>Definicija</b>		Referentni broj		Broj serije		Upotrijebiti do		Gornja granica temperature	<b>RxOnly</b>	Oprez: Savezni zakon ograničava prodaju ovog uređaja na stomatologa ili po nalogu stomatologa.		Ovlašteni predstavnik za Europsku zajednicu		Proizvođač		Europska oznaka sukladnosti		Držati podalje od sunčeve svjetlosti		Pogledajte upute za uporabu
<b>Simbol</b>	<b>Definicija</b>	<b>Simbol</b>	<b>Definicija</b>																						
	Referentni broj		Broj serije																						
	Upotrijebiti do		Gornja granica temperature																						
<b>RxOnly</b>	Oprez: Savezni zakon ograničava prodaju ovog uređaja na stomatologa ili po nalogu stomatologa.		Ovlašteni predstavnik za Europsku zajednicu																						
	Proizvođač		Europska oznaka sukladnosti																						
	Držati podalje od sunčeve svjetlosti		Pogledajte upute za uporabu																						

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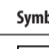
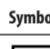
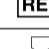
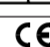
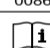
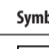
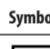
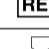
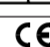
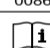
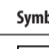
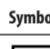
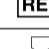
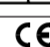
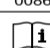
<p><b>Preparation of Other Surfaces</b></p> <p>First, air abrade all bonding surfaces of restorations being cemented or repaired using Danville's MicroEtcher™ and 50 micron Aluminum Oxide. Next, follow instruction related to surface type: Composites, Porcelain and Lithium Disilicate - Apply 5-Bond™ according to its instructions and continue at Step 3.</p> <p>Zirconia, Alumina, and Metals - Apply Z-Bond™ according to its instructions. Surface is ready for composite or cement.</p> <p>Amalgam - Begin at Self-Etch Step 1, above.</p> <p>Precious and Non-Precious Metals - Begin at Step 3. Tin-Plating of precious metal bonding surfaces improves bond strength.</p> <p><b>Adhesive Application (Follow Steps 1 and 2 and continue at Step 3 below.):</b></p> <ol style="list-style-type: none"><li><b>Apply Adhesive</b> with a new MicroBrush™. <b>Scrub for 10 seconds.</b></li> <li><b>Evaporate solvent</b> with gentle air stream. Increase air pressure to <b>thin Adhesive.</b></li> <li><b>Light Cure 10 seconds.</b> Light cure is optional when using Dual/Self-Cure Link with self-cure and dual-cure composites.</li></ol> <p><b>For direct restorations:</b> Place composite (flowable preferred) or compomer restorative and light cure.</p> <p><b>For indirect restorations:</b> See "Preparation of Other Surfaces." Follow instructions below for using Dual/Self-Cure Link on every surface where Adhesive has been applied.</p>	<p>DEUTSCH</p>
---	----------------

**Dual/Self-Cure Link for bonding self-cure and dual-cure composites and resin cement.**

- Dab Link liberally onto cured or uncured Adhesive with a MicroBrush™. Do not scrub.
- Evaporate solvent with air syringe.
- Apply self-cure or dual-cure composite.
- Initiator in self/dual-cured composites will cause Prelude to become dual-cure.

*Warning: Dual/Self-Cure Link may shorten the working time of some self-cure and dual-cure composites.*

<p><b>Definitions of Symbols</b></p> <p>The following symbols may appear on the product packaging or labeling.</p> <table> <tbody><tr> <td><b>Symbol</b></td> <td><b>Definition</b></td> <td><b>Symbol</b></td> <td><b>Definition</b></td></tr> <tr> <td></td> <td>Reference number</td> <td></td> <td>Lot number</td></tr> <tr> <td></td> <td>Use by</td> <td></td> <td>Upper limit of temperature</td></tr> <tr> <td><b>RxOnly</b></td> <td>Caution: Federal law restricts this device to sale by or on the order of a dentist.</td> <td></td> <td>European Community Authorized Representative</td></tr> <tr> <td></td> <td>Manufacturer</td> <td></td> <td>European Mark of Conformity</td></tr> <tr> <td></td> <td>Keep away from sunlight</td> <td></td> <td>Consult instructions for use</td></tr> </tbody></table>	<b>Symbol</b>	<b>Definition</b>	<b>Symbol</b>	<b>Definition</b>		Reference number		Lot number		Use by		Upper limit of temperature	<b>RxOnly</b>	Caution: Federal law restricts this device to sale by or on the order of a dentist.		European Community Authorized Representative		Manufacturer		European Mark of Conformity		Keep away from sunlight		Consult instructions for use	<p>www.zestdent.com</p> <p><b>ZD ZEST DENTAL SOLUTIONS</b></p> <p>Danville Materials  2875 Loker Avenue East  Carlsbad, CA 92010 USA  (1)760-743-7744</p> <p> MDSS GmbH  Schiffgraben 41  30175 Hannover  Germany</p> <p> 90988 Rev J</p>
<b>Symbol</b>	<b>Definition</b>	<b>Symbol</b>	<b>Definition</b>																						
	Reference number		Lot number																						
	Use by		Upper limit of temperature																						
<b>RxOnly</b>	Caution: Federal law restricts this device to sale by or on the order of a dentist.		European Community Authorized Representative																						
	Manufacturer		European Mark of Conformity																						
	Keep away from sunlight		Consult instructions for use																						

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<p><b>Preparation of Other Surfaces</b></p> <p>First, air abrade all bonding surfaces of restorations being cemented or repaired using Danville's MicroEtcher™ and 50 micron Aluminum Oxide. Next, follow instruction related to surface type: Composites, Porcelain and Lithium Disilicate - Apply 5-Bond™ according to its instructions and continue at Step 3.</p> <p>Zirconia, Alumina, and Metals - Apply Z-Bond™ according to its instructions. Surface is ready for composite or cement.</p> <p>Amalgam - Begin at Self-Etch Step 1, above.</p> <p>Precious and Non-Precious Metals - Begin at Step 3. Tin-Plating of precious metal bonding surfaces improves bond strength.</p> <p><b>Adhesive Application (Follow Steps 1 and 2 and continue at Step 3 below.):</b></p> <ol style="list-style-type: none"><li><b>Apply Adhesive</b> with a new MicroBrush™. <b>Scrub for 10 seconds.</b></li> <li><b>Evaporate solvent</b> with gentle air stream. Increase air pressure to <b>thin Adhesive.</b></li> <li><b>Light Cure 10 seconds.</b> Light cure is optional when using Dual/Self-Cure Link with self-cure and dual-cure composites.</li></ol> <p><b>For direct restorations:</b> Place composite (flowable preferred) or compomer restorative and light cure.</p> <p><b>For indirect restorations:</b> See "Preparation of Other Surfaces." Follow instructions below for using Dual/Self-Cure Link on every surface where Adhesive has been applied.</p>	<p>NEDERLANDS</p>
---	-------------------

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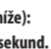
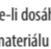
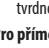
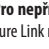
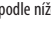
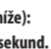
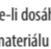
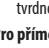
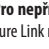
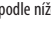
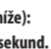
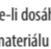
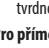
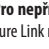
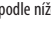
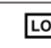
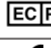
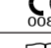
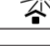
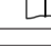
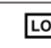
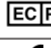
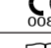
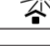
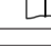
<p><b>Preparation of Other Surfaces</b></p> <p>First, air abrade all bonding surfaces of restorations being cemented or repaired using Danville's MicroEtcher™ and 50 micron Aluminum Oxide. Next, follow instruction related to surface type: Composites, Porcelain and Lithium Disilicate - Apply 5-Bond™ according to its instructions and continue at Step 3.</p> <p>Zirconia, Alumina, and Metals - Apply Z-Bond™ according to its instructions. Surface is ready for composite or cement.</p> <p>Amalgam - Begin at Self-Etch Step 1, above.</p> <p>Precious and Non-Precious Metals - Begin at Step 3. Tin-Plating of precious metal bonding surfaces improves bond strength.</p> <p><b>Adhesive Application (Follow Steps 1 and 2 and continue at Step 3 below.):</b></p> <ol style="list-style-type: none"><li><b>Apply Adhesive</b> with a new MicroBrush™. <b>Scrub for 10 seconds.</b></li> <li><b>Evaporate solvent</b> with gentle air stream. Increase air pressure to <b>thin Adhesive.</b></li> <li><b>Light Cure 10 seconds.</b> Light cure is optional when using Dual/Self-Cure Link with self-cure and dual-cure composites.</li></ol> <p><b>For direct restorations:</b> Place composite (flowable preferred) or compomer restorative and light cure.</p> <p><b>For indirect restorations:</b> See "Preparation of Other Surfaces." Follow instructions below for using Dual/Self-Cure Link on every surface where Adhesive has been applied.</p>	<p>DEUTSCH</p>
---	----------------

<p><b>Dual/Self-Cure Link for bonding self-cure and dual-cure composites and resin cement.</b></p> <ul style="list-style-type: none"><li>Dab Link liberally onto cured or uncured Adhesive with a MicroBrush™. Do not scrub.</li> <li>Evaporate solvent with air syringe.</li> <li>Apply self-cure or dual-cure composite.</li> <li>Initiator in self/dual-cured composites will cause Prelude to become dual-cure.</li></ul> <p><i>Warning: Dual/Self-Cure Link may shorten the working time of some self-cure and dual-cure composites.</i></p> <p><b>Definitions of Symbols</b></p> <p>The following symbols may appear on the product packaging or labeling.</p> <table> <tbody><tr> <td><b>Symbol</b></td> <td><b>Definition</b></td> <td><b>Symbol</b></td> <td><b>Definition</b></td></tr> <tr> <td></td> <td>Reference number</td> <td></td> <td>Lot number</td></tr> <tr> <td></td> <td>Use by</td> <td></td> <td>Upper limit of temperature</td></tr> <tr> <td><b>RxOnly</b></td> <td>Caution: Federal law restricts this device to sale by or on the order of a dentist.</td> <td></td> <td>European Community Authorized Representative</td></tr> <tr> <td></td> <td>Manufacturer</td> <td></td> <td>European Mark of Conformity</td></tr> <tr> <td></td> <td>Keep away from sunlight</td> <td></td> <td>Consult instructions for use</td></tr> </tbody></table>	<b>Symbol</b>	<b>Definition</b>	<b>Symbol</b>	<b>Definition</b>		Reference number		Lot number		Use by		Upper limit of temperature	<b>RxOnly</b>	Caution: Federal law restricts this device to sale by or on the order of a dentist.		European Community Authorized Representative		Manufacturer		European Mark of Conformity		Keep away from sunlight		Consult instructions for use	<p>DEUTSCH</p>
<b>Symbol</b>	<b>Definition</b>	<b>Symbol</b>	<b>Definition</b>																						
	Reference number		Lot number																						
	Use by		Upper limit of temperature																						
<b>RxOnly</b>	Caution: Federal law restricts this device to sale by or on the order of a dentist.		European Community Authorized Representative																						
	Manufacturer		European Mark of Conformity																						
	Keep away from sunlight		Consult instructions for use																						

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