

Contact Wedge™

Elastic Wedge System

Unique elastic wedges provide complete cervical adaptation and seal. Hard wedges provide force at only one point and can dent matrices.

Ideal for use with all sectional matrix systems!

Placement is fast and easy with rubber dam forceps or ring pliers. Interproximal relief feature allows stretching to a very thin profile to 'floss' through tight contacts, yet rebounds back fully. Three color-coded heights are available.

- Create ideal cervical seal
- Fast & easy to use
- Eliminate ledging



Contact Wedge Kit
Small, Medium, & Large
(85 each)
Ref # 91015

Refills (85)
Small
Medium
Large

Ref # 91016
Ref # 91017
Ref # 91018

Contact Matrix™ & Contact Wedge Technique



1 Metal matrix placed interproximal of teeth to be restored



2 The elastic wedge is held in place and achieves contour and separation as band wraps prepped area



3 A MegaRing is placed against the matrix to secure contact and duplicate shape of the tooth to be restored



4 Knotted floss is tied to the ring as a precaution against any problems associated with detachment



5 Completed addition of the ring



6 When necessary two MegaRings are easily placed to ensure proper clinical contact



7 Restoration is complete with ideal adaptation achieved



8 For super separation, two MegaRings may be applied simultaneously