Product Description

Mega V™ represents the innovation behind the converging tines technology and the superior clinical performance only a sectional matrix system can provide. The use of heat-treated stainless steel ensures the intended force is delivered to separate the adjacent tooth, freeing the clinician to perform class II restorations, quickly and efficiently.

Indications for Use

The Mega V sectional matrix system offers a single ring size intended for molars and premolars, which can be stacked to achieve multiple restorations at one time.

Precautions

- For safety, isolate the area with a rubber dam.
- If overspread, the rings may splay. Splay may be corrected by crimping the ring with pliers.
- Rings can be autoclaved at 132°C for 15 minutes. Let cool before using.
- Rings may discolor after autoclaving cycles. This does not affect the quality or function of the ring or tines.
- Matrices are single use only and should be discarded after use.

Instructions for Use:

- Select and place matrix and wedge. Two sizes of matrices are provided.

Note: To open a tight contact, the ring may be placed a few minutes in advance, removed and then the matrix and ring can be placed as usual.

- Expand and place Mega V using Danville's ring pliers or rubber dam forceps.

- Place tines against the matrix and "V" directly over the wedge.

Note: Rings can be easily stacked for multiple restorations. Multiple rings also can be placed in opposite directions.

- Finish your class II restoration.

Definitions of Symbols

The following symbols may appear on the product packaging or labeling.

Symbol	Definition	Symbol	Definition
	Reference number		Lot number
	Use by		Manufacturer
	Caution: Federal law restricts this device to sale by or on the order of a dentist.		European Community Authorized Representative
	European Mark of Conformity		Consult instructions for use
	Do not reuse		

Danville Materials
2875 Loker Avenue East
Carlsbad, CA 92010 USA
(1)760-743-7744
www.zestdent.com

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