

DANVILLE

A-SILICONE SUPER ADHESIVE

Tray Link

Tray Link is a newly developed adhesive for disposable plastic impression trays based on polycarbonate. It assures reliable adhesion of addition curing vinyl silicone impression materials to disposable plastic impression trays and totally avoids lifting of the impression material from the tray while taking the impression out of the mouth.

Applied on the impression tray, the Tray Link layer is not sticky and works even when used hours later.

NOTE

Tray Link works only with addition curing vinyl silicones, such as Danville's Star VPS, 1st Quarter, or First Half impression materials.

APPLICATION

Brush a thin layer of Tray Link on those areas of the tray to be in contact with VPS using an applicator brush. Allow to dry for 30 seconds.

Load the tray and take the impression. After setting of the impression material, it is strongly adhered to the impression tray.

ATTENTION

Close bottle immediately after each use to avoid solvent evaporation!

WARNING

Tray Link should only be used in well ventilated rooms. In case of contact with skin, wash off with water and soap.

STORAGE

Do not store above 25 °C (78 °C)



Danville Materials
a Zest Anchors, LLC company
2875 Loker Avenue East
Carlsbad, CA 92010 USA
(1)760-743-7744

www.zestdent.com

ZD ZEST DENTAL
SOLUTIONS



MDSS GmbH
Schiffgraben 41
30175 Hannover
Germany

