| | | |
|-----------------------------|--------------------|---------------|
| Instructions For Use | First Half™ | RxOnly |
|-----------------------------|--------------------|---------------|

PRODUCT DESCRIPTION

First Half™ is a super hydrophilic, fast setting vinyl polysiloxane impression material. The setting time is slower than Danville’s super-fast First Quarter™ materials but faster than the Start VPS™ impression materials. First Half is odorsless and tasteless. It is immisible in glutaraldehyde solutions and other disinfectants. First Half materials are compatible with all other vinyl polysiloxane materials.

INDICATIONS

First Half is an additional-cure vinyl polysiloxane dental impression material that is used for all crown and bridges, edentulous and implant techniques.

WORK AND SET TIMES

| First Half | Color | Working Time At Room Temperature (23°C) min:sec | Setting Time Intraorally (35°C) min:sec |
|-----------------|------------|---|---|
| Light Body | Black | 1:00 | 2:15 |
| Light Body | Orange | 1:00 | 2:15 |
| Monophase Ortho | Purple | 1:00 | 2:15 |
| Heavy Body | Green | 1:00 | 2:15 |
| Putty | Light Blue | 1:00 | 3:00 |

Note: Work and set times may vary depending on storage conditions and temperature.

CARTRIDGE PREPARATION

1. Insert cartridge into an appropriate 1:1 mixing gun, remove twist-off cap, and extrude a small amount of material until even flow from both barrels is seen. Discard the dispensed material and wipe the cartridge end clean with tissue, avoiding cross contamination of the barrels.

2. Attach an automatic tip of desired size. The material is now ready for use.

3. After use, do not remove the automix tip. The used tip serves as a seal until removed for the next use.

CLOSED BITE IMPRESSIONS

1. Select an appropriate tray. ***(Technique Tip: It is best to avoid metal trays "bite trays" if a second pour is made in the laboratory. Metal trays tend to distort when the lab pries the first cast out of the impression, precluding an accurate second pour.)***

2. The impression should be made by using both the Heavy Body and Light Body simultaneously. Generally one person loads the tray with the Heavy Body while another person syringes Light Body onto the teeth. Then the tray is seated over the Light Body. Sometimes it is helpful to blow air on the Light Body to get it to flow onto the teeth, prior to seating the tray. To avoid distortion, it is important to fully seat the impression before any elasticity develops in the impression materials. ***(Technique Tip: Putty should not be used in place of the Heavy Body since it tends to result in elastic distortion)***

- Please refer to the table above for work and set times. Since setting time depends strongly on the temperature of the impression material, it is wise to check the setting of some extra material left at room temperature before retrieving the impression. (Generally setting in the mouth is faster due to higher temperature, creating a margin of safety.)
- Water wash and dry the impression after removal from the mouth.

SEPARATE FULL ARCH "PUTTY/WASH" IMPRESSIONS (use Light Body and Start VPS Putty)

Creating the most accurate impressions using putty requires a putty pre-polymerization technique. The putty is allowed to fully polymerize (and fully cure) before using the wash material to provide accuracy and detail. First Half Light Body is ideal for the wash, especially with the Danville needle tip attached to the automix tip.

- Before preparing teeth, make a putty impression in a retentive tray such as a rim lock or a perforated tray. Prevention of interproximal tags and space for the wash is easily created by applying Reynolds Wrap or a section of a sandwich bag to the putty before seating in the mouth. Do not remove the plastic wrap until ready for the wash. ***(Technique Tip: some brands of plastic wrap will interfere with the polymerization of the putty, leaving a very obvious liquid putty surface. A test is advised before use.)***
- For the final impression, remove the plastic wrap and apply Light Body to the putty surface. Optionally dispense Light Body onto the clean and dry prepared teeth. Seat the impression tray.
- Please refer to the table above for work and set times. Since setting time depends strongly on the temperature of the impression material, it is wise to check the setting of some extra material left at room temperature before retrieving the impression. (Generally setting in the mouth is faster due to higher temperature, creating a safety margin.)
- Water wash and dry the impression after removal from the mouth.

CUSTOM TRAY TECHNIQUE

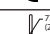
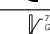
The custom tray is usually made of methyl methacrylate or light-polymerizing material such as Triad™. First Half Light is ideal as a wash material. A vinyl polysiloxane adhesive must be applied to the tray and dried before the wash is placed. Follow the adhesive manufacturer’s instructions. Light Body is applied as for the putty/wash technique.

ADDITIONAL NOTES

- First Half materials should be brought to room temperature prior to use. Exposure to prolonged temperatures above 77°F can be damaging. Store at room temperature.
- First Half materials are compatible with all other vinyl polysiloxane materials.
- Powder from gloves can impair set. Sample test is suggested. Keep putty dry closed when not in use.
- High viscosity materials used alone are not suitable for detailed impressions.
- Light Body Black impression materials used alone can fix excessively and may result in distortion.
- Procedures and techniques prepared courtesy of Raymond Bertolotti, DDS, PhD. For further information, please contact 5th Quarter Seminars at (510) 483-2411, Fax (510) 652-8729. www.adhesion.com

DEFINITIONS OF SYMBOLS

The following symbols may appear on the product packaging or labeling.

| Symbol | Definition | Symbol | Definition |
|---|---|---|--|
|  | Reference number |  | Lot number |
|  | Use by |  | Manufacturer |
| RxOnly | Caution: Federal law restricts this device to sale by or on the order of a dentist. |  | European Community Authorized Representative |
|  | European Mark of Conformity |  | Consult instructions for use |
|  | Upper limit of temperature |  | Keep away from sunlight |

| | | | | |
|---|---|---|---|---|
|  | Danville Materials 2875 Loker Avenue East Carlsbad, CA 92010 USA (1)760-743-7744 |  | MDSS GmbH Schiffgraben 41 30175 Hannover Germany |  |
|---|---|---|---|---|

| | | |
|---|------------------|-------------|
|  | www.zestdent.com | 90746 Rev G |
|---|------------------|-------------|

| | |
|--------------------|-----------------|
| First Half™ | HRVATSKI |
|--------------------|-----------------|

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