S Bond™ Instructions for Use	ENGLISH
---	----------------

PRODUCT DESCRIPTION

S Bond™ is a simple and effective one component pre-hydrolyzed silane primer for porcelain and ceramic restorations. It requires no mixing or other activation. Typically S Bond is used on a laboratory etched porcelain or composite restoration to link the restoration to a bonding resin.

INDICATIONS FOR USE

S Bond is a highly stable, pre-hydrolyzed silane. It provides full strength bond link to silica ceramics. Use on etched porcelain and composite restorative materials.

INSTRUCTIONS FOR USE

Apply S Bond to a clean, dry surface directly from the bottle by touching the restoration to the bottle tip, or with a brush. Dry with clean air. Apply a resin bonding agent (e.g. Prelude One™, Danville Materials) to the S Bond primed surface according to the manufacturer’s instructions for use.

If you wish to prepare a porcelain or glass surface for bonding that has not been hydrofluoric acid (HF) etched:

A) etch with HF per manufacturer’s instructions for use. Apply S Bond per the instructions above.

or

B) roughen the surface with a sandblaster (e.g. MicroEtcher, Danville Materials) using aluminum oxide abrasive. If sandblasting is unavailable, use a coarse diamond bur. Wash with 37% phosphoric acid, rinse with water, and dry with air. Apply S Bond per the instructions above.

STORAGE

Shelf life is 2 years. Store at room temperature (23°C; 73°F). Cap tightly to prevent loss of volatile components. Refrigerated storage will extend the shelf life.

DEFINITION OF SYMBOLS

The following symbols may appear on the product packaging or labeling.

SYMBOL	TITLE	EXPLANATORY TEXT	STANDARD	REFERENCE
	Manufacturer	Indicates the medical device manufacturer, as defined in EU Directive 93/42/EEC	EN 980 ISO 15223-1	5.12 5.1.1
	Authorized Representative in the European Community	Indicates the authorized representative in the European Community	EN 980 ISO 15223-1	5.13 5.1.2
	Catalogue Number	Indicates the manufacturer’s catalogue number so that the medical device can be identified	EN 980 ISO 15223-1	5.10 5.1.6
	Batch Code	Indicates the manufacturer’s batch code so that the batch or lot can be identified	EN 980 ISO 15223-1	5.4 5.1.5
	Consult Instructions for Use	Indicates the need for the user to consult the instructions for use	EN 980 ISO 15223-1	5.18 5.4.3
	Use-by date	Indicates the date after which the medical device is not to be used	EN 980 ISO 15223-1	5.3 5.1.4
	Upper Limit of Temperature	Indicates the upper limit of temperature to which the medical device can be safely exposed	EN 980 ISO 15223-1	5.17.2 5.3.6
	Keep Away from Sunlight/heat	Indicates a medical device that needs protection from light sources	EN 980 ISO 15223-1	5.20 5.3.2
	European Mark of Conformity	Indicates device is in conformance with Medical Device Directive 93/42/EEC	MDD 93/42/EEC	Annex XII
RxOnly	Rx Only	Caution: Federal law restricts this device to sale by or on the order of a dentist	US Code of Federal Regulations, Title 21	801.15(c)(1)(i)(F)

	Danville Materials 2875 Loker Avenue East Carlsbad, CA 92010 USA (1)760-743-7744 www.zestdent.com		MDS5 GmbH Schiffgraben 41 30175 Hannover Germany		89105-00 REV F
		RxOnly			

S Bond™ Upute za uporabu	HRVATSKI
---	-----------------

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