ENGLISH

INSTRUCTIONS FOR USE
CHARISURE® Attachment Processing Material

Indications
CHARISURE® Attachment Processing Material is a dual cure (either self-cure or UV light cure) tissue colored composite material that is used to press attachments full and partial dentures either a chair-side or laboratory procedure. It may also be used to fill voids in denture.

Advantages
• Low viscosity, delivered in a double barrel syringe
• Self cure as accelerated curing with the application of UV light
• Long working time
• The material bonds to itself to allow for easy filling or addition of material.

Important – Special Instructions
Federal USA I strictly advise this use to be on the order of a dentist or properly licensed practitioner. The remote possibility exists that a patient may have sensitivity towards CHARISURE® Attachment Processing Material. Should an allergy reaction occur, the material must be avoided and an alternative material with different chemistry should be employed.

CHARISURE® Attachment Processing Material is not affected by the MR environment and therefore has not been tested for heating or migration in the MR environment.

Single Use Only
The mixing tray of the CHARISURE® Attachment Processing Material is for single use only. If re-used, Zest Anchors cannot guarantee the functionality or the safety of the product. CHARISURE® Attachment Processing Material cannot be dispensed properly with re-used mixing tips.

Application
1. Denture Preparation: Being that CHARISURE® Attachment Processing Material does not bond to another acrylic, an additional step in the attachment is needed to provide maximum mechanical retention for the attachment to the denture.
2. CHARISURE® Attachment Processing Material Preparation: Remove the end cap on the syringe or cartridge by turning it 1/8 turn counterclockwise and pulling. Discard the cap.
3. When using a syringe or cartridge for the first time, express material from the syringe or cartridge without a mixing tip. Use a mixing tip if desired. Mix until the mixture is a uniform color. Do not use the mixing tip to dispense the material.
4. Attach a new mixing tip by rotating the mixing tip to stop on the syringe end and rotate 1/8 turn clockwise to lock the tip in place. If it is desired to utilize the Angled Tip, snap it into the end of the mixing tip.

5. Black or any undyed unshaded anterior attachments, abutments or teeth.
Note: After the denture is checked, any undenture that was not blocked out sufficiently may cause the denture to be locked into place.

6. Dispense the ZEST CHARISURE® Attachment Processing material. Apply a small amount of material around the attachment in the mouth or on the model. Dispense additional material until excesses in the denture. The working time for CHARISURE® Attachment Processing Material is 1 minute. After the material is dispensed into the time the material is expressed from the tip. Be sure to seat the denture passively, without excessive biting pressure. If the patient is allowed to displace the tissue by dosing function, the attachments may be in the wrong position in the denture. This will make it difficult to seat the denture and have the attachments provide the proper retention.

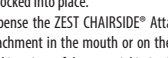
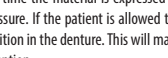
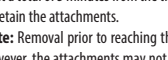
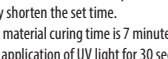
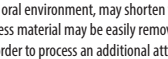
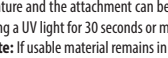
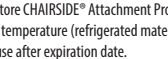
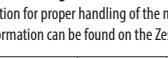
7. Wait 1 to 5 minutes from the time the material is first mixed to set. At this point the material is self-sufficiently to retain the attachments.

8. Excess material may be easily removed from areas of the denture using a dental bur.
9. In order to process an additional or alternate attachment to the denture, remove the mixing tip. Replace it with an unused tip. Change previous attachment per the previous steps. Any voids detected between the denture and the attachment can be filled outside the mouth or away from the model. The material may be cured using a UV light for 30 seconds or may be allowed to self cure for a minimum of 7 minutes.

Note: If isokule material remains in the syringe, always leave the stop off attached to the syringe.

Storage
Do not store CHARISURE® Attachment Processing Material Processing material above 25° C/77°. Only use oral temperature (refrigerated material is more viscous and cures more slowly). Do not use after expiration date.

Material Handling
Information for proper handling of the material may be found on the Material Safety Data Sheet (MSDS). This information can be found on the Zest Anchors website (www.zestanchors.com).

Symbol	Standard	Explication/Significado de símbolo
	ENISO 12523-1	Símbolo para "BATCH CODE" The batch/code label shall be adjacent to the symbol.
	ENISO 12523-1	Símbolo para "MANUFACTURER" The symbol shall be adjacent to the manufacturer's name.
	ENISO 12523-1	Símbolo para "CATALOGUE NUMBER" The product catalog number shall be after or below the symbol adjacent to it
	ENISO 12523-1	Símbolo para "CONSULT INSTRUCTIONS FOR USE"
	ENISO 12523-1	Símbolo para "NON-STERILE"
	N/A	CE Marking of Conformity (Use 0066) only on class I or higher risk European Authorized Representative
	N/A	Prescription Required.
	ENISO 12523-1	Símbolo para "USE ONLY" This symbol shall be adjacent to a date to indicate that the device should not be used after the end of the useful month shown.

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